

Mobile Acute Cart for Phoenix Dialysis Machine



The Mobile Acute Cart combines the Gambro's Phoenix Dialysis Machine and our PA2000 Reverse Osmosis System on a lightweight, high clearance, stable, caster base for maneuvering through hospital corridors and lifts.



Applied Water Engineering, Inc.

The cart can be pushed or pulled from the water end. When pushing, the thrust on the handle is transferred to the handle grip on the rear of the Phoenix.

The Phoenix is fully accessible on the cart and the rear shelf is easily removed or reinstalled. There is no need to remove from cart for normal maintenance.



Nevertheless, it is easy to take the Phoenix off the cart and put on another. A wooden ramp allows 1 person loading and unloading.



The Phoenix wheelbase is confined on the cart with sidewalls. It is held securely in place with a SS threaded band clamp. The only modification necessary to the Phoenix is cutting out some of the foamed plastic that wraps around its metal frame base for the band clamp.

A nominal 6x36 tank of Centaur™ GAC is mounted next to the RO. This allows the system to be used in areas with high chloramine and reduces cartridge filtration replacement costs.

Overall size is 24" wide by 44" long.



Applied Water Engineering, Inc.

2759 South 300 West, Suite G

Salt Lake City, UT 84115

Phone 801-486-7774

Fax 801-484-9265

awewater.com

